IT

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EN

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated. The brace is softly padded and is therefore extremely comfortable to wear. The integrated anatomically preformed aluminum splint ensures an optimal fit.

Instructions:

The brace is anatomically pre-formed upon delivery. The fit can be optimised by bending the integrated aluminium stabilisation elements to match the patient’s characteristics. This may only be carried out by the specialist orthopaedic dealer / physician. To apply the brace, pull it over the wrist and position the thumb in the thumb hole. Ensure that the circular position encompassing the thumb lies flat and does not cause any pressure points. Now close the circular closure encompassing the thumb and then close the wide closure located on the wrist. The tightness is infinitely adjustable thanks to the Velcro fastener.

The brace can be re-adjusted over and over again during the treatment in order to achieve an ideal fit thanks to the formable aluminium stabilisation element. The brace can be used in water without any problems whatsoever. Rinse with plenty of freshwater should the brace come into contact with saltwater.

Indications:

Post-traumatic, post-operative, i.e. following tendon reconstruction, immobilisation in the case of degenerative processes such as de Quervain’s Disease, for inflammatory and rheumatic diseases, such as rhizarthrosis, tendopathy.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

38 % polyamide-velour, 28 % cotton, 28 % polyurethane, 6 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor’s order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor. If pressure points, abrasions or similar occur, have your orthopaedic technician or doctor check the fit of the support immediately.

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