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EN

Immobilisation is a basic principle in treating injuries and overloading as it speeds up the healing process.

This orthosis has soft padding so is very comfortable to use. The integrated stabilising element guarantees an excellent fit.

Instruction for use:

The orthosis is anatomically pre-shaped when supplied. For an excellent fit, the orthosis must be adapted to the needs of the patient by bending the integrated, aluminium stabilising element. This operation should only be carried out by the orthopaedic store or by a doctor.

To put on correctly, slip the orthosis over the wrist and insert the thumb through the special opening. Ensure that the circular structure rests on the surface of the thumb without causing any pressure points. First close the circular fastener on the thumb, followed by the fastener on the wrist.

Thanks to the Velcro the fastener can be adjusted as necessary. Thanks to the deformable aluminium stabilising element it is always possible to adjust the orthosis throughout the course of treatment to obtain an excellent fit.

The orthosis can be used in water. In the event of contact with saltwater, rinse with plenty of clean water after use.

Indications:

Post-traumatic, post-operative, for example after thumb fractures, arthritis, arthrosis of the thumb saddle and/or metacarpophalangeal joints, ulnar/radial ligament injury to the thumb, late stage after sinew reconstruction, tendopathy, for controlled mobilisation in the case of rhizarthrosis, ski thumb.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

55 % polyurethane, 45 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

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