IT

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EN

Wrist Supports support injured structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements.

The enclosed soft elastic Stabilo® band can be individually adjusted. Due to the thumb opening the support does not slip.

The plastic splint offers optimal support. The highly elastic form-knit allows easy support application and yet provides a good supporting effect.

Instructions for use: Adaptation of the anatomically pre-shaped plastic splint: If required, the material of the plastic splint can be individually adapted by the orthopaedic technician at a temperature of about 180 °C. Allow the reshaped splint to cool and then re-insert into the splint pocket of the support.

Indications: Chronic post-traumatic or post-operative irritations in the wrist area, i.e. sprains, carpal arthrosis, tenosynovitis, chronic, post-traumatic or post-operative irritations in the area of saddle joint of the thumb, e.g. in the case of ulnar and radial ligament instability, early stages of rhizarthrosis.

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

57 % polyamide, 28 % elastodienes/latex, 8 % spandex, 7 % viscose

Important Note: No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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