IT

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Wash by hand in hand warm water with light-duty detergent. Close Velcro fastening to avoid damage of other clothes. Stretch in form, air-dry. Avoid heat.

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EN

The BORT Patella Tendon Strap supplies pressure onto the tendon which is connected to the knee cap (patella). This supports the patella during movements with the knee to slide in the correct course. Simultaneously the pressure provides increased muscular strength, which is important for the guidance of the patella.

Instruction for use:

Open the strap completely and place it directly below the patella. Slip Velcro strap through the D-ring and close Patella Tendon Strap in a way that pressure is applied to the patella tendon, however vascular congestion to the lower leg is avoided.

Indications:

Patellar chondropathy, femoropatellar arthrosis, patellar tendonitis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

65 % polyamide, 35 % elastane

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear strap during night, avoid hindering blood circulation. In case of numbness loosen strap and remove it if necessary. If complaints continue, please contact the doctor.

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