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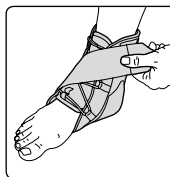
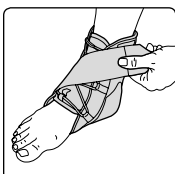
EN**Instruction for use:**

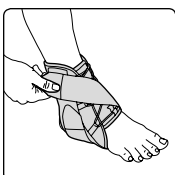
Lacing can be carried out when putting it on using the hook and eyelet elements or optionally through the eyelets or the laces can be attached to the hook.

Close the ankle brace so that the support fits closely but causes no congestions. The Stabilo® strap which is provided offers two modes of use.

**In case of subinjury:**

Type of injury: subinjury trauma, twisting of ankle with strain of the outer ligament system of ankle.
Aim: Lifting of the outer foot side (Stabilo® strap pulls up the outer foot side).

In order to do so, the Stabilo® strap is fastened in the medial malleolus area, is guided underneath the sole of the foot towards the outer side of the foot whilst pulling slightly and is finally diagonally guided past the back of the foot towards the inner side of the support and secured at this point.

**In case of pronation:**

Type of injury: pronation trauma, twisting of ankle with strain of the inner ligament system of ankle.
Aim: Lifting of the inner foot side (Stabilo® strap pulls up the inner foot side).

In order to do so, the Stabilo® strap is fastened in the lateral malleolus area, is guided underneath the sole of the foot towards the inner side of the foot whilst pulling slightly and is finally diagonally guided past the back of the foot towards the outer side of the support and secured at this point.

Indications:

Post-operative protection after ligament suture / reconstruction, conservative treatment of ankle distortions and ligament ruptures, immobilisation in the case of inflammatory irritations, rheumatism, arthritis, tendinitis, ligament injuries.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

56 % polyamide, 28 % polyester, 16 % spandex

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.
In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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