IT

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EN

Knee Supports support injured tissue structures (joint, ligaments, capsules, cartilages). Thus the compression supports the fixing function of the ligaments and also releases the joint. At the same time the support helps to a high degree to avoid mismovements.

The pad has a massage effect which reduces the tendency for swelling and helps to recede existing swelling more quickly. It is a further advantage of the pad that the patella is being guided by the strips during the normal movements.

Indications:

Patellar chondropathy, meniscopathy, arthritis, bursitis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

57 % polyamide, 28 % elastodien, 8 % spandex, 7 % viscose

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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