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EN

Knee braces support damaged tissue structures (joints, ligaments, capsules, cartilage). Compression aids the supporting function of the ligaments and reduces the load on the joint. At the same time, to a large extent the band helps to avoid incorrect movements.

The splints assist the stabilising function of the lateral ligaments.

Instruction for use:

Before wearing the knee brace, the joint splints must be adjusted by a specialist with training in body shaping.

The knee brace has a joint for the inner part of the leg (medial) and for the outer part of the leg (lateral).

To shape, remove the fastening straps from the joint and take the splints out of the knee brace.

In order to limit extension/flexion, the included mounting devices can be used. The extension can be limited at 10°, the flexion at 30°, 60° and 90°.

Now the joint splints can be shaped to fit the shape of the leg, so that they remain flat and do not create pressure points.

The splints are then put back into their pockets and the straps adjusted, without being fully closed.

Now pull the knee brace onto the leg, so that the silicone opening is centred on the kneecap (patella).

Firstly fasten the leg straps, then those on the thigh. Now the knee brace should rest comfortably on the leg, without any pressure points.

Slight corrections can be made to the shape of the splints without having to remove them.

Caution: If the splints cause pressure and redness, remove the knee brace and have it checked by a healthcare professional.

Indications:

Meniscus injuries, mild to moderate lateral ligament instability of the knee joint, gonarthrosis, arthritis, bursitis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

57 % polyamide, 28 % elastodies, 8 % spandex, 7 % viscose

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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