IT

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**EN**

Knee Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements. The hinges are beneficial to the stabilisation function of the lateral ligaments. The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restrain of swelling. A further advantage is that the patella is by the pad being positioned into normal motions.

#### Application of the support:

Open the Velcro straps first, tighten the support so that the pads enclose the kneecaps. Fix the support with a wide Velcro fastener, then attach the remaining 4 Velcro fasteners so that the support supports and compresses but doesn't constrict.

**Indications:** Mild to moderate lateral ligament instability of the knee joint, arthrosis, arthritis.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

63 % polyamide, 35 % elastodien, 2 % elasthan

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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