IT

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EN

Knee Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements.

Instructions for use:

First open Velcro fastener, then put on the support in a way that the pad embrace the patella. Finally close both Velcro closures in a way that the support supports and is compressing but not constricting.

Indications:

Recurrent articular effusion, arthrosis, arthritis, ligament instability.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

40 % cotton, 40 % elastodies, 20 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor’s order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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