IT

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**EN**

Knee Support support injured tissue structures (joints, ligaments, capsules, cartilages). The compression supports the stabilisation function of the ligaments and thus relieves the joint. The combination of both silicone pads in connection with a velour strap provides localised pressure relief of the tibial tuberosity ligament with relaxation of the patellar tendon via the u-shaped silicone pad with its opening facing downwards.

**Instructions for use:**

Slip the support over the forward section of the foot and then pull it up to the knee. Here, the upper arc of the silicone half ring should be placed directly above the kneecap. The lower silicone pad then surrounds the tibial tuberosity ligament (shin roughness). Following this, attach the correction strap by gradually moving it upwards onto the velour area.

**Indications:** Facilitation of the movement of the patella, Osgood-Schlatter disease, tendinosis of the tibial tuberosity ligament, gonarthrosis, arthritis, articular effusion, irritations.

**Contraindications:** Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

**Composition:** 57 % polyamide, 28 % elastodien, 8 % spandex, 7 % viscose

**Important Note:**

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.  
 While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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