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EN

Knee Supports support injured tissue structures (joint, ligaments, capsules, cartilages). Thus the compression supports the fixing function of the ligaments and also releases the joint. At the same time the support helps to a high degree to avoid mismovements. The BORT Asymmetric® guides and stabilises the kneecap through a lateral/proximal silicone half-ring.

Instructions:

1. Slip the support over the end of the foot and then pull the support up to the knee. Position the BORT Asymmetric® on the knee joint in such a manner that the kneecap (patella) is laterally supported by the silicone half-ring.
 2. First detach the upper bracing element from the Velcro closure and reattach whilst pulling slightly towards the inner side of the leg.
 3. Then secure the lower bracing element below the knee cap accordingly whilst pulling slightly toward the inner side of the leg.
- It is recommended to apply the BORT Asymmetric® whilst seated.**
4. Please detach the Velcro closure before removing the BORT Asymmetric®. It is not possible to remove the support otherwise and this may also lead to the support becoming damaged.

Indications: Patellofemoral pain syndrome, high riding patella, patella lateralisation, recurrent or permanent post- surgery lateral release condition, recurrent or habitual patella instability, meniscopathy.

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition: 61 % polyamide, 36 % elastodien, 3 % viscose

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.
In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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