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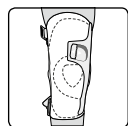
**EN**

Knee Support support injured tissue structures (joints, ligaments, capsules, cartilages). The compression supports the stabilisation function of the ligaments and thus relieves the joint. Swelling tendencies are reduced, existing swelling recedes faster. At the same time the support helps to avoid false movements. Both medial as well as lateral side joint splints stabilise the knee joint. Adjustable pull straps fix the joint splints to the thigh and lower leg. The joint splints can be adjusted to match the contour of the leg if required. They can be removed from the splint pockets in order to carry this out. The BORT Asymmetric® Plus guides and stabilises the kneecap through a lateral/proximal silicone half-ring.

#### Instructions:

Fully open all circular Velcro closures.

**Important:** It is recommended to apply the BORT Asymmetric® Plus whilst seated.


#### Step 1

Slip the support over the end of the foot and then pull the support up to the knee. Position the BORT Asymmetric® Plus on the knee joint in such a manner that the kneecap (patella) is laterally supported by the silicone half-ring.

#### Step 2

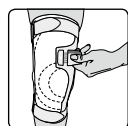Now fasten the circular pull straps on the thigh and lower leg.

#### Step 3

Now detach the small Velcro closure located above the patella, pull it down to the inner side of the leg whilst applying slight force and secure it back onto the Velcro surface located on the base support.

#### Step 4

Please detach the Velcro closure before removing the BORT Asymmetric® Plus. It is not possible to remove the support otherwise and this may also lead to the support becoming damaged.

#### Indications:

High riding patella, patella lateralisation, patellofemoral pain syndrome, Patellar luxation or subluxation, post-operative.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

Knitted fabric: 65 % polyamide, 28 % elastodien, 7 % viscose

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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