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## EN BORT EasyLife foldable Gripping Aid

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

**Intended purpose**  
This aid is a gripper or a "helping hand" which extends the arm, enabling its user to grip objects.

**Indications**  
Paraplegia and restricted movement, for example.

**Contraindications**  
Muscular failure/Lower arm region or hand muscle dysfunction/ Nickel allergy.

**Application risks/Important notes** ⚠  
This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom

you have received the medical device will select the appropriate size and instruct you regarding its use.

- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not grip in case of open palm wounds
- do not use in case of intolerance of one of the materials used, e.g. nickel
- no re-use – this medical aid is intended for treating one patient
- be careful when gripping fragile or sensitive items

**Operating instructions**  
Grasp the blue hand-held device and operate the moveable lever using your fingers. If you press it, the gripper closes and if you apply no pressure it opens again. Depending on the surface quality/ form, a maximum of some 1.5 kg can be lifted. The BORT EasyLife foldable gripper weighs approx. 260 g and can provide an extension of approx. 75 cm.  
For removal or transport, you can collapse the gripper (approx. 45 cm).

- 1 Hold the rear area of the gripper firmly and fold the front area upwards.
- 2 Then hold the part pushed upwards firmly and fold the rear area upwards.
- 3 Finally, join both parts together fully in order to prepare the gripper for transport.

To fold it open, press the top and the bottom parts downwards again. You can lock the gripping aid in a closed status.

4 Press the lever back as far as it goes and press the button on the end of the grip. To release, press the lever back as far as it goes again.

5 The leverage can be set from the grip component as follows: Press the lever back as far as it goes and take a sharp object (e.g. a biro) and move the bar to the desired position:  
V Position: Longer leverage – lower exertion of force in case of restricted manual gripping ability.  
✓ Position: Shorter leverage – normal exertion of force in case of unrestricted manual gripping ability.

**Removal**  
Please store the gripper at room temperature.

## Material composition

Pipe: Aluminium  
Catch/locking device, reverser: POM plastic (polyoxymethylene)  
Grip, bottom part, intermediate part: ABS plastic (acrylonitrile butadiene styrene)  
Springs: Chrome/nickel

**Cleaning information**  
The BORT EasyLife foldable gripping aid can be washed using mild detergents or manually with a damp cloth and may not be dried mechanically.

**Guarantee**  
The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

**Useful life/Lifetime of the product**  
The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

**Duty of notification**  
If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the appointed authority for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

**Disposal**  
Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

**Declaration of conformity**  
We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

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