BORT EasyLife Greifhilfe Standard

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REF 118 150

REF 118 170

REF 118 190

D118150|2024-02|004 ML | Rev. 01

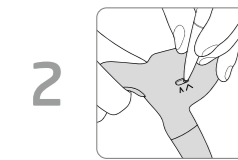
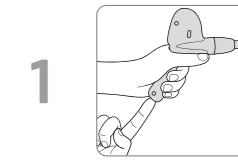
4 005862 149160

- DE BORT EasyLife Greifhilfe Standard**
Gebrauchsanweisung beachten | Medizinprodukt
Einzelner Patient – mehrfach anwendbar
- EN BORT EasyLife Standard Gripping Aid**
Observe the instructions for use | Medical device
Single Patient – multiple use
- FR BORT EasyLife Aides à la préhension Standard**
Suivez les instructions d'utilisation | Dispositif médical
Un seul patient – à usage multiple
- ES BORT EasyLife Pinza de presión estándar**
Siga las instrucciones de uso | Producto sanitario
Un solo paciente – uso múltiple
- IT BORT EasyLife Pinza prensile standard**
Seguire le istruzioni per l'uso | Dispositivo medico
Singolo paziente – uso multiplo
- NL BORT EasyLife-grijphulp standaard**
Gebruiksaanwijzing in acht nemen | Medisch hulpmiddel
Eén patiënt – meervoudig gebruik
- CS BORT EasyLife Uchopovací pomůcky Standard**
Dbejte návodu k použití | Zdravotnický prostředek
Jeden pacient – vícenásobné použití
- ET BORTi haaramise abivahend EasyLife**
Järgige kasutusjuhendit | Meditsiiniseade
Ühel patsiendil korduvalt kasutatav

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Illustrationen Figures

DE BORT EasyLife Greifhilfe Standard

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Stand: 02.2024

EN BORT EasyLife Standard Gripping Aid

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This aid is a gripper or a "helping hand" which extends the arm, enabling its user to grip objects.

Indications

Paraplegia, restricted movement.

Contraindications

Muscular failure/Lower arm region or hand muscle dysfunction/ Nickel allergy.

Application risks/Important notes

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
– use the medical device in accordance with therapeutic needs

– only use other products simultaneously after consultation with your physician
– do not make any changes to the product
– do not grip in case of open palm wounds
– do not use in case of intolerance of one of the materials used, e.g. nickel
– no re-use – this medical aid is intended for treating one patient
– be careful when gripping fragile or sensitive items

Operating instructions

Grasp the blue hand-held device and operate the moveable lever using your fingers. If you press it, the gripper closes and if you apply no pressure it opens again. Depending on the surface quality/form, a maximum of some 1.5 kg can be lifted.

You can lock the gripping aid in a closed status.

1 Press the lever back as far as it goes and press the button on the end of the grip. To release, press the lever back as far as it goes again.

The leverage can be set from the grip component as follows:

2 Press the lever back as far as it goes and take a sharp object (e.g. a biro) and move the bar to the desired position:
V Position: Longer leverage – lower exertion of force in case of restricted manual gripping ability.
✓ Position: Shorter leverage – normal exertion of force in case of unrestricted manual gripping ability.

REF	Extension length (approx.)	Weight (approx.)
118 150	50 cm	220 g
118 170	75 cm	240 g
118 190	90 cm	260 g

Removal

Please store the gripper at room temperature.

Material composition

Pipe: Aluminium
Catch/locking device, reverser: POM plastic (polyoxymethylene)
Grip, bottom part: ABS plastic (acrylonitrile butadiene styrene)
Springs: Chrome/nickel

Cleaning information

The BORT EasyLife gripping aid standard can be washed using mild detergents or manually with a damp cloth and not be dried mechanically.

Garantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

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