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**EN**

Immobilisation is a basic principle in the treatment of injuries and overstrain since it greatly accelerates the recovery process and prevents secondary complications. Shoulder-arm supports support damaged tissue structures (joints, ligaments, capsules, cartilage). The BORT OmoTex Shoulder Traction Brace comes with four 250-gram weights. These provide an additional means of adjusting the upper arm traction. The weights are placed individually into a removable pocket on the brace to enable a customized weighting and the weight cuff can be applied according to the course of therapy even without the compression support.

#### Instruction for use:

**Important:** Support must be removed before radiological examinations (e.g. MRI, X-ray, CT)!

The BORT OmoTex Shoulder Traction Brace should be placed on the patient by a health professional. Slip the brace over the upper arm and the injured shoulder until the head of the humerus is properly positioned in the space provided. Place the elastic belt down over the head and the healthy arm through the strap so that the strap runs diagonally across the chest and under the opposite shoulder without cutting in at the neck. The brace can also be worn with both parts of the strap running behind the upper body to the opposite shoulder. Slip the uninjured arm through the strap so that it runs around the opposite shoulder and across the back. Use the Velcro fastener to tighten the elastic belt as desired. The inside two-piece silicone pad can be adjusted by means of the Velcro fastener. This is used to support and harness the head of the humerus from the front and the rear. To remove the weight cuff from the shoulder support pull the fastener tape from the loop and remove the Velcro. Place the weight cuff around the upper arm and fasten it with fastener tape. The weight elements can be used to relieve any painful tightening below the shoulder (subacromial decompression).

#### Indications:

Impingement syndrome, chronic post-traumatic or post-operative irritations of soft tissues in the shoulder area, humeroscapular periarthropathy (HSP).

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

Support: 65 % polyamide, 28 % elastodies, 7 % viscose  
Weights: Filling: 100 % steel, pocket: 80 % PA, 20 % spandex

#### Important Note:

No reutilization – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor. If pressure points, abrasions or similar occur, have your orthopaedic technician or doctor check the fit of the support immediately. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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