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EN

Immobilisation is a basic principle in treating injuries and overloading as it speeds up the healing process. The BORT OmoFX Vario orthosis for abduction of the shoulder and arm makes it possible to comfortably fix the shoulder and elbow joints with an abduction angle of optionally 30° or 45°. The removable lining makes the orthosis very comfortable to wear, as well as hygienic.

Instruction for use:

First of all place the injured arm on the foam cushion and use the three Velcro strips to ensure that it is secured firmly to the cushion. Then bring the tape positioned at the height of the arm from the back to the front, passing over the shoulder. Bring the tape back towards the cushion, through the guide ring and fasten the Velcro. Take the tape positioned at waist height and wrap around the back, thread through the guide ring and fasten. If necessary, pull both tapes so that the orthosis fits snugly against the body.

Indications:

Pre-operative, post-operative, post-traumatic, for example in the case of impingement syndrome, rotator cuff rupture, implantation of shoulder prosthesis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

44 % polyamide, 30 % ester-based polyurethane, 26 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor’s order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

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