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EN

Immobilisation is a basic principle of the treatment of injuries and strains, as thus the recovery process will be considerably accelerated.

Shoulder/arm abduction braces support damaged tissue structures (joints, ligaments, capsules, cartilage), whilst at the same time permitting a functional movement of uninjured structures.


The BORT OmoARS Short Shoulder Abduction Brace allows you to comfortably fix the shoulder joint and elbow at an abduction angle of around 15°.

Instructions for use:

The initial fitting and adjustment is usually performed in the operating room.

Patients should always sit at a table when putting on and removing the brace. Doing so will help to prevent any harmful shoulder movements.


Tip: Open the Velcro straps on the front of the pad that are used to close the sling. Also open the Velcro strap on the hip belt at the front of the brace pad.

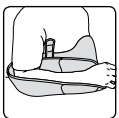
Place your injured arm in the open sling so that your hand is resting on the ball.

Make sure your elbow is well-positioned in the sling and then close the sling.

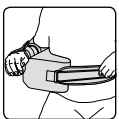
Pull the hip strap from behind your back around to the front and fasten the Velcro strap.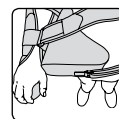

For the optimal functional position of the Omo ARS (15° abduction angle), position the cushion so that the front surface is parallel to the front of your body.

Tip: If you are facing a table, the front surface of the cushion must be parallel to the edge of the table.

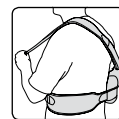Pull the shoulder strap forward over your uninjured shoulder.

Fasten the Velcro strap.

Indications: Pre-operative, post-operative, post-traumatic, for example in the case of impingement syndrome, rotator cuff rupture, implantation of shoulder prosthesis, subacromial defilee dilation (acc. to Neer), subacromial bursitis.

Contraindications: Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition: 43 % polyamide, 33 % PUR foam, 24 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

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