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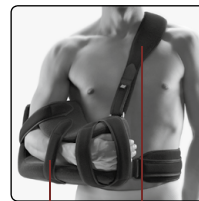
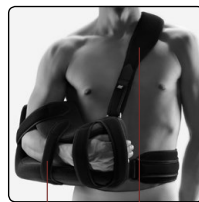
EN

Immobilisation is a basic principle of the treatment of injuries and strains, as thus the recovery process will be considerably accelerated.

By means of the BORT OmoFX Shoulder-Arm Abduction Brace shoulder joint and elbow can be fixed in a comfortable way at an angle of approx. 30°.

The removable cover cares for pleasant, hygienic wearing features.

Instructions for use:

strap 2 strap 1

First put the injured hand into the half-open foamed shell as shown in the picture and position it safely by means of the three narrow Velcro straps.

Then lead the strap (1), which is provided at the height of the upper arm, from the rear over the shoulder. Lead the strap through the D-ring at the thumb, reverse it and fasten the Velcro closer. Lead the second strap (2) at waist height around the back, lead it through the D-ring near the little finger and fasten the Velcro closure. Put on strap (1) and (2) in such a way, with possible re-tightening, that the brace rests tightly at the body.

Indications:

Pre-operative, post-operative, post-traumatic conditions, for example in the case of periarticular irritations and disorders in the glenohumeral joint area, acromioclavicular joint dislocation (Tossy Classification Grades I + II), supraspinatus syndrome.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

44 % polyamide, 30 % ester-based polyurethane, 26 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

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