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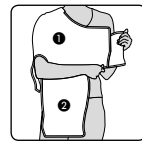
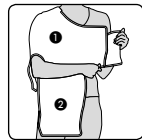
EN

Immobilisation is a basic principle in the treatment of injuries and overstrains, as the healing process is significantly accelerated and consequential injuries are prevented.

Shoulder-Arm Abduction Braces provide support to damaged tissue structures (joints, ligaments, capsules, cartilage) while at the same time allowing undamaged structures to be exercised.

The integrated hand rest in the form of a foam ball allows the lower arm muscular system to be exercised at an early stage.


Instructions for use:


1. Depending on whether you are going to wear the vest on the left or right shoulder, turn the vest so that the shoulder pad shaped like a half-circle (1) is positioned on the side of the body you would like to immobilise and the forearm support (2) is hanging in front of your body.

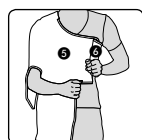
2. Since the support can be worn on both the left and right side, please cover the Velcro strips facing outward by folding the material over. For easier handling, please fasten belt 3 to belt 4. This will make it easier for you to attach belt 7.

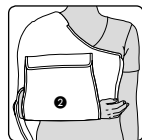
3. Close the support over your chest by fastening the section of the support marked "6" to the section marked "5".

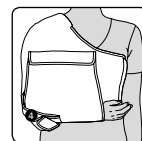4. Bend your arm on the afflicted side at the elbow (usually at a 90° angle), then close the forearm support (2) toward your head.

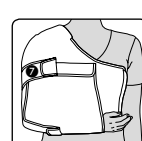5. Close the Velcro strip on the elbow support (4) below your elbow so that your elbow rests comfortably and securely in the intended space.

6. Finally, fasten the remaining Velcro strip (7), located at upper-arm level, toward the front of your body.

Indications:

Pre-operative, post-operative, post-traumatic, for example in the case of sprains, dislocations, subcapital humerus fractures, injury to the rotator cuff.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

65 % PES foam, 35 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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