# BORT OmoBasic nach Gilchrist

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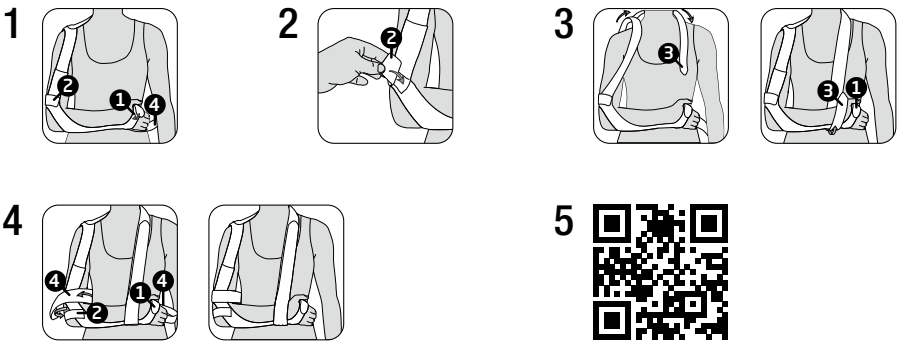
REF 121 350

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D12135012023-071008 ML | Rev. 01

## DE BORT OmoBasic nach Gilchrist

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## EN BORT OmoBasic acc. to Gilchrist

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

### Intended purpose

This medical device is a shoulder joint brace for immobilisation of the shoulder joint in a defined position.

### Indications

Pre-operative, post-operative, post-traumatic, e.g. in case of distortion, luxation, subcapital humerus fracture, rotator cuff injury, scapula fracture.

### Contraindications

Lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

### Application risks/important notes

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- if wearing at night is prescribed, avoid negative influence on the circulatory system
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material.

### Fitting instructions

**1** The lower arm is angled at 90° in the elbow joint and placed in the wide lower arm support. Then, belt 1 is closed on the back of the hand, enabling the individual setting of the metacarpus width. **Important!** The further-reaching belt 4 here is placed on the ventral side on the palm.

**2** Belt 2 is then set to the elbow width and can be shortened using a pair of scissors if required.

**3** Belt 3 is laid over the shoulder from the sewed-on belt origin and guided to the opposite shoulder over the neck. Then, guide belt 3 beneath the lower arm support on the stomach side at wrist level and re-attach it to the wrist using Velcro. This belt distributes the weight on the shoulders and retains the 90° position.

**4** Belt 4 runs from the origin of the palm circumferentially around the waist and is guided ventrally around the arm affected and attached to the same belt using Velcro.

**5 Tip:** You can find our video, which shows the attachment process, both on our homepage and via the product-specific QR code.

### Removal

To remove the OmoBasic acc. to Gilchrist brace, open all the Velcro fastenings in the reverse order and then remove the brace.

### Material composition

Polyamide (PA), PUR foam (PUR), polyester (PES)

The sewn in textile label provides the precise material composition.

### Cleaning information

Delicate wash 30 °C Do not bleach Do not dry in a tumble dryer  
 Do not iron Do not clean chemically  
Close the Velcro fastening to avoid damage to other laundry items.  
Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

### Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

### Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

### Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

### Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

### Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

Status: 07.2023

Medical device | Single patient – multiple use

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