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EN

Immobilisation is a basic principle for treatment of injuries and overstrains, as it considerably accelerates the recovery process, preventing consequential damages.

Shoulder arm braces relieve affected tissue structures (joints, ligaments, capsules, cartilage).

With the BORT OmoBasic® shoulder brace acc. to Gilchrist, the shoulder joint and elbow can be rested in adduction position to the body in a very comfortable and safe way, thus enabling easy wound inspection.

Instructions for use:

The BORT OmoBasic® acc. to Gilchrist should be put on by a specialist.

1. The brace should be put on with the elbow bent by 90°. Lead the hand through the hand loop, which can be adjusted to the correct width by means of a Velcro closure.

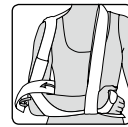
2. Now put the forearm rest on, so that the forearm is positioned in the brace, and the blue straps surround the elbow joint laterally and rest against the upper arm. The width of the brace on the upper arm can be adjusted by means of a Velcro closure.


3. As the next step, position the shoulder part at shoulder height and guide the sewed-on shoulder strap around the neck to the front.

4. To stabilise the forearm, guide the shoulder strap below the wrist, turn it around upwards and close it with the Velcro fastener.

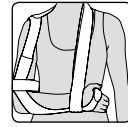5. Finally, guide the body strap, which is fastened at the hand section, around the back towards the arm, which shall be immobilised, and there between the upper arm and body again to the front side and fix the end with the Velcro fastener onto the body strap positioned at the back.

- If required, the hand can additionally be fixed using the attached narrow, black strap.

Indications:

Pre-operative, post-operative, post-traumatic, for example in case of sprains, dislocations, subcapital humerus fractures, scapular fracture, rotator cuff injury.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

43 % polyamide, 33 % PUR foam, 24 % polyester

Important Note:

No reutilisation – this medical device is designed for the care of one patient only.

In case a doctor prescribes wearing the brace at night, avoid hindering blood circulation. In case of numbness, loosen the brace and remove it if necessary. If complaints continue, please contact the doctor.

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