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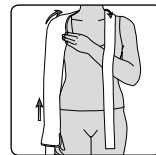
**EN**

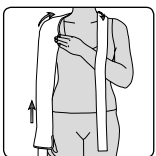
Immobilisation is a basic principle for treatment of injuries and overstrains, as it considerably accelerates the recovery process, preventing consequential damages.

Shoulder arm braces relieve affected tissue structures (joints, ligaments, capsules, cartilage).


With the closed form BORT OmoBasic®, you can place your shoulder joint and elbow comfortably and securely in the adducent position close to your body.

### Instructions for use:


A specialist should fit the closed form BORT OmoBasic® for you.

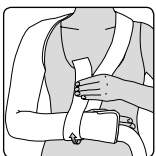
1. Place your hand on the injured arm through the tube's opening and initially pull the tube upwards until the tips of the fingers are visible. Hold the tube securely at the wrist and with your other hand and pull the brace upwards toward the shoulder joint. When doing so, be careful not to pull too firmly on the shoulder joint. Then, wind the shoulder belt around the shoulders and neck back towards the front of the body.

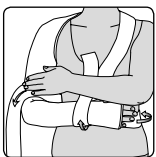2. Bend the injured arm at the elbow at a 90° angle, unless otherwise directed. To stabilize the forearm, wrap the shoulder belt under the wrist and back upwards, and then fasten it using the Velcro strip.

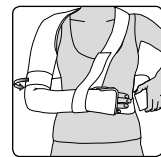3. Wind the body belt, which is attached to the hand section away from the hand, around the back, and towards the arm which is being immobilized. Then, run the belt forward between the body and forearm, around the upper arm, and then around the back of the body.

4. Securely fasten the end of the belt to a section of body belt already wrapped around the body.

### Indications:

Pre-operative, post-operative, post-traumatic, for example in case of sprains, dislocations, subcapital humerus fractures, scapular fracture, rotator cuff injury.

### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

### Composition:

49 % cotton, 23 % Pur-ester, 22 % polyamide, 4 % elastodiene, 2 % polyester

### Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case a doctor prescribes wearing the brace at night, avoid hindering blood circulation. In case of numbness, loosen the brace and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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