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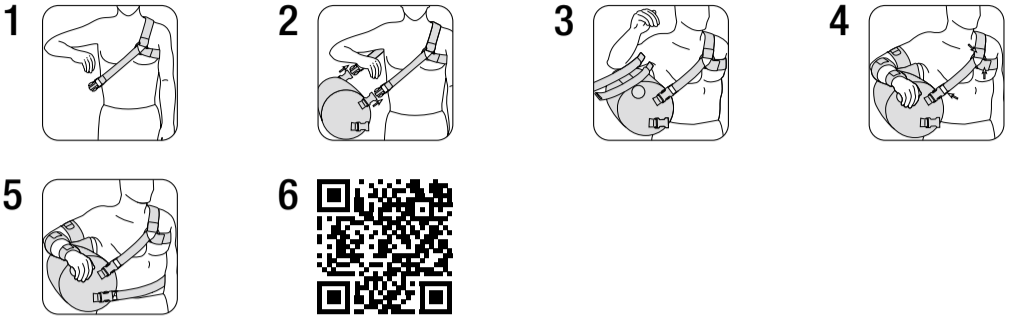
**REF** 121 800

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D121800I2023-10I008 ML I Rev. 01

### DE BORT OmoTwinAir

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### EN BORT OmoTwinAir

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

#### Intended purpose

This medical device is a shoulder joint brace for immobilisation of the shoulder joint in a defined abduction position.

#### Indications

Pre-operative, post-operative, post-traumatic, e.g. in case of impingement syndrome, rotator cuff tear, endoprosthesis implantation.

#### Contraindications

Lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

#### Application risks / Important notes

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- if wearing at night is prescribed, avoid negative influence on the circulatory system
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the support, please neither use any creams nor ointments on or around it as they can destroy the material

#### Fitting instructions

The BORT OmoTwinAir is available in a universal size and can be used both on the right and left sides. Fill both air cushion chambers with air using the hand pump provided. The BORT OmoTwinAir shoulder-arm abduction brace allows very comfortable fixation of the shoulder joint and elbow at an abduction angle of approx. 50°.

- 1 Place the strap system around the uninjured shoulder. For this, guide a short harness over shoulder height, the second short harness through below the axilla and attach them to the plastic rings. Position the plastic rings so that they are placed in the shoulder blade region on the back and in the chest region at the front.
- 2 Place the air-filled cushion under the arm and connect it to the strap system.

3 In order to fix the lower arm on the cushion in the position required for therapy, the lower arm loops with integrated hand rests can be freely positioned on the cushion using Velcro fasteners.

4 Then, attach the wedge-shaped upper arm support to the cushion. Observe that incorrect positioning of the upper arm support can negatively effect the therapeutically correct positioning. Then close both fasteners on the lower arm as well as the upper arm fastener. The strap system Velcro fasteners enable individual setting of the desired resting position.

5 The waist strap, which additionally fixes the cushion to the body, serves for additional fixing.

6 **Tip:** You can find our video, which shows the attachment process, both on our homepage and via the product-specific QR code.

#### Removal

To remove the brace, open the fasteners and take it off.

#### Material composition

PUR foam (PUR), polyamide (PA), cotton (CO), polyester (PES) The sewn in textile label provides the precise material composition.

#### Cleaning information

Delicate wash 30 °C Do not bleach Do not dry in a tumble dryer Do not iron Do not clean chemically Close the Velcro fastening to avoid damage to other laundry items. Before washing, remove the cover. Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

#### Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

#### Useful life / Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

#### Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

#### Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

## Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

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