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EN

Immobilisation is a basic principle for treatment of injuries and strains, as thus the recovery process will be considerably accelerated and consequently damages prevented.


Shoulder-arm-abduction braces support injured tissue structures (joints, ligaments, capsules, cartilages) and allow simultaneously functional training of uninjured structures.

By means of the BORT OmoTwinAir Shoulder-Arm-Abduction Brace the shoulder joint and elbow can most comfortably be fixed with an abduction angle of about 50 °.


Instructions for use:

The BORT OmoTwinAir is available in one universal size, applicable for the right as well as also for the left side.


Fill up with air the 2 chambers of the air cushion with the attached manual pump.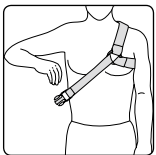


Put the strapping system around the uninjured shoulder. For that lead a short strap over the shoulder height, the second short strap underneath the axillary and fasten these to the plastic rings in such a way, that these are placed in the area of the shoulder blade at the dorsum, and in the front in the area of the chest.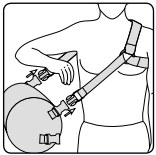


Put the air loaded cushion under the arm and connect it to the strapping system.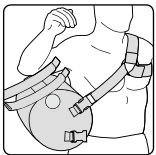

In order to fix the forearm on the cushion in the therapeutically required position, the forearm loops can be freely positioned by means of a Velcro closure.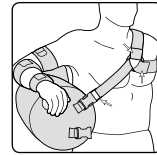

Now fix the wedge-shaped upper armrest onto the cushion. Pay attention that by a false positioning of the upper armrest the therapeutically correct resting can be negatively influenced. Now close both closures at the forearm as well as the upper arm closure. By means of the Velcro closures at the strapping system the desired resting position can be individually be adjusted.

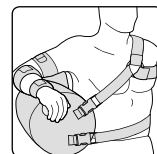The waist strap, which fixes the cushion additionally to the body, serves as supplementary fixation.

By deflation of air out of both air chambers the abduction in the shoulder joint can be reduced.

To avoid an undesired anteversion of the shoulder joint when lying, air can be deflated of the rear chamber only. The rear chamber is color-marked at the valve.

Indications:

Pre-operative, post-operative, post-traumatic, for example in the case of impingement syndrome, rotator cuff rupture, implantation of shoulder prosthesis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

44 % polyamide, 30 % Ester-based polyurethane, 26 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

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