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EN

Immobilisation is a basic principle of treating injuries and overuse symptoms as it significantly speeds up the recovery process and prevents further complications. BORT OmoSAT supports injured tissue structures (joints, ligaments, capsules, cartilages).

Instructions for use:

Place your forearm into the arm sling and put the strap system around your uninjured shoulder. This is done by pulling the strap from the front of your body over your shoulder to your back. At the back, you can attach the strap to the arm sling. Fix your arm in the desired position using the Velcro closures of the strap system. The waist strap is an additional aid to hold your arm securely against your body.

Indications:

Pre-operative, post-operative, post-traumatic, for example in the case of sprains, contusions, subcapital humerus fractures, signs of paralysis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

43 % polyamide, 33 % PUR foam, 24 % polyester

Important Note:

Not reusable – this orthopedic aid is intended for use by one patient only. If you were instructed to wear the sling at night, make sure it does not hamper your blood circulation.

If you feel any numbness, loosen the sling or take it off if necessary. Contact your physician if symptoms persist.

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