IT

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**EN**

This is a special support for the so called tennis elbow. Pain in the elbow will occur through overstraining. Application of this brace 2" below the elbow joint of the forearm will reduce the tension in this area.

#### Instructions for use:

To position the pad correctly, please take care of the marking, printed on the brace. The narrow part of pad should always show in the direction of elbow joint. Thread the Velcro strap into plastic ring and then slip the brace onto your forearm. The inserted pads are to exert pressure on the muscle elevation from the back of the hand to the elbow. You can feel this muscle by squeezing your fist. The pressure of the BORT Epicondylitis Strap should only be felt on this muscle when you squeeze your fist or grasp something with your hand. If the hand is re-laxed, the brace should fit without any pressure

being felt from the pads. Close the Velcro fastening to make the brace fit tightly without hindering the blood circulation. Cut off the overlapping Velcro strap.

#### Indications:

Epicondylitis, chronic or recurrent epicondylopathy, tendopathies, overstraining, so-called 'tennis-/golf elbow'.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

**Composition:** 74 % polyamide, 26 % polyurethane

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only. Wear brace only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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