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EN

Injuries in the cervical vertebra area are affecting a complex structure of ligaments, muscles, sinews, nerves and vessels. Immobilisation support recovery of these tissue structures and helps at the same time avoiding to a high degree mismovements.

Instruction for use:

Open up cervical support and grip it with both hands horizontally. The Velcro closure strap is placed at the back. Put on cervical supports in a way that the recess fits directly below the chin. Then wrap the support around the neck and lock the Velcro closure at the rear side. Place the cervical support in a way that it supports but does not strangle. The support is made of soft foam and therefore can also be worn at night.

Indications:

Art. nos. 127260, 127360, 127660, 127460:

Pain in the lumbar spine region, Slight whiplash, Sprains, Cervical syndrome, Post-operative.

Art. nos. 127560, 127570:

Severe pain, Moderate whiplash, Cervical syndrome, Torticollis (rheumatic), Osteochondrosis/spondylarthrosis of the lumbar spine, Post-operative.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

PUR foam, cover 100 % cotton

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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