IT

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**EN**

Immobilisation is a basic principle for the treatment of injuries and overstress, as the curing process will be significantly accelerated.

#### **Instructions for use:**

The splint comprises 3 pieces: a large center part and two Velcro-fixed lateral elements. The lateral elements can be freely positioned and allow for adjustment to various circumference dimensions. Place the splint from the rear around the leg, in a way that the patella will be surrounded by the opening. The lateral elements should overlap by approx. 2 cm. Feed Velcro closures through the plastic eyelet, revers and fix Velcro closure. Put on the splint in a way that the knee joint is certainly immobilised, however no vascular congestion in calf and leg area is caused.

#### **Indications:**

Pre-operative, post-operative, post-traumatic, joint, muscle or ligament injuries.

#### **Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

#### **Composition:**

35 % polyamide, 35 % cotton, 30 % PUR foam

#### **Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.

Wear splint only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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