IT

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EN

Immobilisation is a basic principle for the treatment of injuries and over-stress, as the curing process will be significantly accelerated.

The one-piece knee splint is 55 cm long and has 2 separate splint pockets with Velcro fixing. In each of them is one pliable aluminium rail. The special design of this product enables the model being done in one circumference size only.

Instructions for use:

Fit the Immobilisation Splint from the rear around the leg in a way that the dorsal positioned aluminium rail is located in the middle of the hollow of the knee (poplitea). Feed the Velcro closures through the plastic D-rings, turn over, pull back and fix it. Position the splint in a way that the knee joint is of course immobilised, however no vascular congestion in calf and foot area is caused.

Put on the splint in a way that the knee joint is certainly immobilised, however no vascular congestion in calf and leg area is caused.

Indications:

Pre-operative, post-operative, post-traumatic, joint, muscle or ligament injuries.

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

35 % polyamide, 35 % cotton, 30 % PUR foam

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

Wear splint only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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