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## EN

Lumbar spine brace support damaged tissue structures (joints, ligaments, capsules, cartilage). The circular enclosure supports the holding function of the ligaments and thereby relieves the lumbar spine. The combination of circular compression and stabilisation provided by semi-rigid orthotic half-shells relieves and supports the affected region. The brace also provides increased protection against incorrect movements and offers postoperative and initial conservative securing of the segments 1st sacral vertebra to 4th lumbar vertebra.

### Instruction for use: Putting on the Brace

- Open the two buckles in the front.
- Pull up on the two round Boa fasteners located on each side to unlock them.
- Make sure that there is a distance of around 6 cm between the halves of the pelvic harness. If the harness is too tight, remove some cord from the BOA fasteners to loosen it.
- Fit the brace around your body so that the upper parts of the synthetic elements are positioned above the iliac crest.
- Fasten the elastic brace with the help of the attached loops.
- Check the length of the black straps and adjust then if necessary.
- First fasten the lower black strap and then the upper black strap. Check the brace for a proper fit. You should be able to feel the upper part of the synthetic shell clearly above the iliac crest.
- Now lock the BOA fasteners by pressing in on the dials until they click into place. You can now turn the dials to tighten the brace. While doing so, you should feel the synthetic stabilization elements embracing your body.
- To remove the brace, unlock the BOA fasteners by pulling out on them. Use the black tabs to pull the cord out of the BOA fastener.

### Adjustment to therapy progress

The BORT Stabilo® Lumbar Back Brace can be adapted to suit the therapy progress. The lumbar spinal relief is continuously adjustable (see above). On the dorsal side the degree of delordosation can be adjusted by bending the aluminium splints integrated in the pelvic harness. During the course of therapy it may be necessary to reduce the stabilising effect of the back brace. To do this, remove the stiffer version of the splints from the outer splint pockets on the lumbar side of the abdominal support. The more flexible splints can now be inserted. If stabilisation with the plastic elements is no longer

required, they can be removed from the support which can be worn as a normal back support.

### Indications:

Major lumboischialgia without absolute indication for surgery, when surgery is contraindicated, when surgery is rejected, major radicular and pseudo radicular lumbar syndrome (cautionary, therapy-resistant), spondylolisthesis with severe lumboischialgia, severe lumbar deformity with hypermobility associated with Facet Syndrome / Osteoarthritis, condition after spine surgery, i.e. dorso-ventral/dorso-lateral fusion, spinal column compression (single level with deformity and multi-level), thoracic- surgery with severe prolapse / lumbar instability, spinal canal stenosis with paresis, post-traumatic vertebral fractures with significant spinal damage.

### Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

### Composition:

84 % polyamide, 16 % spandex

### Important Note:

No reutilization – this medical device is meant for the care of one patient only. In case of doctor’s order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

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