IT

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EN

Knee Supports support injured tissue structures (joints, ligaments, capsules, cartilages). The compression supports the stabilisation function of ligaments and thus relieves the joint.

Instructions for use: Before putting the support on, the side joint splints must be adapted to the contour of the leg by an expert. For forming, remove the splints from the textile and align to the lateral course of the patient's leg by bending it up (if applicable using a bending iron). Then, set the extension and flexion restrictions on the medial and lateral toothed segment articulation. On delivery extension is adjusted to 0°, flexion is not limited. A different extension stop may be mounted by loosening the assembly screw, the limit wedge, inserted by the company, to be removed, the desired limitation wedge inserted and fastened with the assembly screw.

In the same way the flexion wedge can be handled. Before putting on, open Velcro straps and support. Put on the support in a way that the patella recess is placed directly around the patella. Subsequently close the support frontal at thigh and shank. Finally tighten both Velcro closures in such a way that the support supports and compresses, but does not indent.

Indications: Moderate ligament instability of the knee joint, meniscus injuries, after-care in arthroscopic procedures involving the knee, chronic, postoperative and post-traumatic inflammations.

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

49 % polyamide, 33 % polyester, 11 % polyurethane, 4 % elastane, 3 % silicone

Important Note: No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

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