IT

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EN

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The open type BORT ManuStabilPro Wrist Support supports the wrist by means of three malleable aluminium splints. Lightly apply brace which can be fully opened. Secure fit through special thumb cut-out.

Instructions for use:

Bend to required shape before fitting the aluminium splints. Open the brace completely and push the thumb through the thumb cut-out. Then close the fasteners on the forearm, starting with the fastener closest to the wrist.

Indications:

Post-operative, post-traumatic conditions, degenerative, e. g. carpal tunnel syndrome (CTS), tenosynovitis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

44 % polypropylene, 43 % polyamide, 14 % polyurethane

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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