IT

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EN

Pain in your buttocks and perineum can be an additional problem, especially when sitting. Here the BORT Codyn® cushion, which has been developed in cooperation with orthopaedists, brings noticeable relief. This seat cushion, which is shaped like a horseshoe, effectively relieves pressure on the buttocks and the entire perineal area.

Instruction for use:

Place the inflated cushion onto the chair, so that the imprinted BORT logo shows to the back of the chair. The cushion should be filled with the amount necessary to relieve pressure on the tailbone. This amount can vary depending on the patient's weight.

Indications:

Coccygodynia, condition after coccyx contusion and fracture, perianal relief of haemorrhoids, after rectal surgery, periproctitic abscess, condition after episiotomy.

Contraindications:

Sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

100 % polyurethane

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.
Do not use for patients with more than 150 kg!

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