BORT StabiloHip Hüftprotektor

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REF 201 100

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D201100I2022-10I007 ML I Rev. 01

DE BORT StabiloHip Hüftprotektor

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EN BORT StabiloHip Hip Protector

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a hip protector for reducing the forces which influence the hip/femoral neck impact absorption in case of a fall.

Indications

Tendency to fall for various reasons.

Contraindications

Unclear soft tissue swellings, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

Application risks/Important notes

- The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
- remove the medical device prior to radiological examinations
 - in case of numbness, loosen the medical device and remove it if necessary
 - in case of persistent complaints, consult the physician or specialist retailer
 - use the medical device in accordance with therapeutic needs
 - only use other products simultaneously after consultation with your physician
 - do not make any changes to the product
 - do not wear it on open wounds
 - do not use in case of intolerance of one of the materials used
 - no re-use. This medical aid is intended for treating one patient only
 - whilst wearing the protector, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

Put the hip protector on in the same manner as a pair of trousers. The pockets integrated in the trousers ensures correct positioning of the protector pads. To remove the protector pads, turn the trousers "inside-out". Then put the upper protector bag cover flap behind the protector pad and pull the protector pad out of the lower section of the bag. To use the protector, fold the upper cover flap outwards in order to push the protector with the wide side first into the lower section of the bag and finally put the cover flap over the narrow end of the foam cushion. Please observe that the concave protector cushion side must point towards the body.

Removal

Remove the hip protector similarly to a pair of trousers.

Material composition

Cotton (CO)
The sewn in textile label provides the precise material composition.

Cleaning information

Delicate wash 60 °C Do not bleach Do not dry in a tumble dryer Do not iron Do not clean chemically
Remove the protector pads before washing. The trousers can be washed at 60°C in the washing machine using mild detergents. Do not use fabric conditioner. Stretch back into shape and dry in the fresh air. Hand-wash the protection pads with lukewarm water and mild detergents.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 10.2022

Medical device | Single patient – multiple use

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