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EN

Hip Protectors absorb resp. distribute the forces affecting the hip in case of tumbling and therefore reduce the risk of a fracture of the femur collar.

The BORT StabiloHip hip protector comprises two coordinated components, the functional protector panty and the protector set made of special, shock-absorbing foam. In combination they result in an effective prevention measure and significant reduction of fracture risk in case of tumbling.

Instructions for use:

The BORT StabiloHip Hip Protector is distinguished by its high wearing comfort and simple handling.

The foam protector cushions cling to the body by its three dimensional shaping and the special features of the used material and thus provide an optimal fit. The high elastic band at waist and leg of the protector panty guarantee easy putting-on and off and high compliance. By the integrated pockets of the panty correct positioning of the protector cushions is ensured.

For removing the protector cushions turn over the panty, then turn the upper flap of the protector pocket behind the protector cushion and pull the protector cushion out of the lower part of the pocket. For insertion of the protector fold forward the flap and slip the protector with the wide side ahead into the lower part of the pocket and finally put the flap over the narrow end of the foam cushion. Pay attention that the concave side of the protector cushion faces the body.

Accessories available:

Art. no. 201 200 Replacement panties

Art. no. 201 120 Protector set

Indications:

Tendency to fall for various reasons.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

95 % cotton, 5 % spandex (LYCRA®)

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

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