E

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ⒼⒷ

Hip Protectors absorb resp. distribute the forces affecting the hip in case of tumbling and therefore reduce the risk of a fracture of the femur collar.

The BORT StabiloHip Protector Belt consists of two matched components, the functional protector belt and the protector set made of special, shock absorbing foam. In combination they result in an effective prevention measure and significant reduction of fracture risk in case of tumbling.

Instructions for use:

The BORT StabiloHip Protector Belt is distinguished by its high wearing comfort and simple handling.

The foam protector cushions cling to the body by its three dimensional shaping and the special features of the used material and thus provide an optimal fit.

The StabiloHip Protector Belt can be worn over and under clothing. The hook loop fastener makes fitting and removing the belt easy. The breathable material prevents the skin perspiring. The pockets integrated in the belt ensure correct positioning of the protector cushions. Note that the concave side of the cushion faces towards the body.

Indications:

Tendency to fall for various reasons.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

79 % Polyester, 21 % spandex

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

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