IT

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EN

Anatomically shaped knee support for stabilisation of knee joint. The V-shaped stabilisation strips attached guarantee a high degree of protection against rotation without disturbing normal motions. The thigh part pulled high up improves muscle function to stabilise the knee. The anti-slip action thanks to the silicone pads on the inside of the upper and lower ends and the anti-crease action thanks to the extra elasticity at the hollow of the knee ensure a high degree of comfort.

Indications:

Chronic post-traumatic or post-operative irritations of knee joint soft tissues, recurrent articular effusion, arthritis, arthrosis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

60 % polyamide, 34 % elastodien, 6 % spandex

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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