IT

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**EN**

Elbow Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. The compressing knit and the silicone pad provide a massage effect, accelerating the healing process. Targeted pressure alleviates pain and relieves traction on the tendon origins.

#### **Instructions for use:**

Apply the support such that the knitted comfort area is positioned in the crook of the arm. The support is properly fitted if you can feel the outer and inner joint extension through the cut-outs of the silicone pad.

#### **Indications:**

Chronic post-traumatic or post-operative irritations of soft tissues in the area of the elbow joint, Epicondylitis (acute or chronic), tendomyopathy.

#### **Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

#### **Composition:**

Colour skin-tone: 58 % polyamide, 17 % polyester, 16 % elastane, 9 % viscose  
Colour mineral grey: 81 % polyamide, 13 % elastane, 6 % viscose

#### **Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

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