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EN

Knee Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements.

The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restraint of swelling.

A further advantage is that the patella is by the pad being positioned into normal motions.

Indications:

Chronic post-traumatic or post-operative irritations of knee joint soft tissues, recurrent articular effusion, i.e. in the case of arthritis, gonarthrosis, ligament instability.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

Colour skin-tone: 54 % polyamide, 26 % polyester, 20 % elastane

Colour mineral grey: 76 % polyamide, 15 % elastane, 9 % viscose

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

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