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INSTRUCTION FOR USE

BORT. Benefit at your side.

Eigenschaften

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Properties

The BORT StabiloColor® Tape is a Kinesiology Tape for sports medicine, therapy and prophylaxis. The BORT StabiloColor® Tape conforms to the human skin in terms of thickness and elasticity and has the following properties:

- Material: 95 % cotton / 5 % elastha
- Stretch: 130 %
- Latexfree
- Skin friendly acrylic adhesive with good adhesive properties
- Good wearing comfort
- Air- and fluid permeable
- Water-resistant

Recommendation for use

The BORT StabiloColor® Tape can be applied directly to the skin with or without tension. We recommend you to apply the BORT StabiloColor® Tape by physicians, physiotherapists or other qualified personnel.

Basis of a each taping application is a medical examination by a medical specialist. In case of persistent discomfort, we recommend you to see a medical specialist. Taping can then only be a complementary form of therapy.

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Indications

- Muscle injury and muscle pain (strain, fibre tear, atrophy, imbalance)
- Joint pain (ligament injury, instability, arthrosis, swelling)
- Chronic overload damage (muscles, ligaments and tendons)
- Mal position and defective position (e.g. Scoliosis)

Contraindications

- malignant tumor, radiation, chemotherapy
- allergy against acrylic adhesive
- acute or chronic skin disease and skin irritation
- vascular pathology such as thrombosis
- open wounds, infections
- temperature of unknown origin

In case of pregnancy, please consult your physician or therapist before using the performance tape.

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Instructions

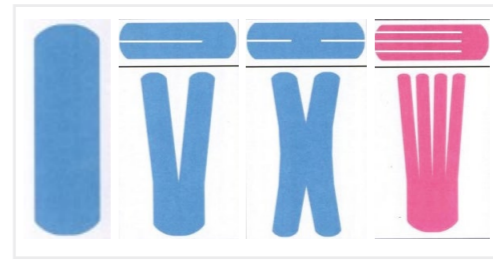
1. Make sure the area is clean, dry and free of lotions and oil
2. Remove excessive body hair
3. Measure and preposition the tapes
4. Cut off the required length of tape and round both ends
5. Tear the protective paper by stretching the tape
6. Apply tape in the applicable technique
7. Do not stretch the ends of the tape
8. Rub the tape firmly to activate the adhesive
9. Apply tape minimum 30 minutes before sporting activity

After taping

The BORT StabiloColor® Tape is intended for use during normal daily activities, including showering, bathing and swimming. In order to prolong wearing time, BORT StabiloColor® Tape can be blow-dried on low level. We recommend to use no cream or body lotion in the designated taping area.

The BORT StabiloColor® Tape can be used up to 10 days and subsequently be replaced by a new tape. BORT StabiloColor® Tape is best removed under shower.

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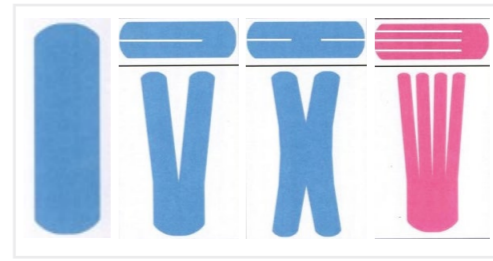
Cutting techniques

Taping techniques

- Muscle technique (in direction of the muscle)
- Ligament technique (improves weight-bearing capacity)
- Corrective technique (mal positions and defective positions)
- Fascia technique (declamps muscle and fascia tension)

For additional information please consult your physician or therapist.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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Examples of use

- Shoulder joint (Y-Tape)
- Neck (Y-Tape with horizontal stripes)
- Lower back (weave form tape with two stripes)
- Adductor (I-Tape)
- Knee joint (round tape with 2 or 4 stripes)
- Ankle joint (combination tape)

Colours

Each colour is made of the same material with identical properties. The BORT StabiloColor® Tape is available in following colors:

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BORT GmbH | apartado de correos 1330 | D-71367 Weinstadt | www.bort.com
BORT GmbH | Case postale 1330 | D-71367 Weinstadt | www.bort.com