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EN

The BORT Valco® SOFT Hallux Valgus Brace serves postoperative and conservative treatment of hallux valgus (large toe malpositioning). The brace can be worn both on bandages and also in sufficiently spacious shoes during the daytime or at night. The padded aluminium reinforcement splint should be shaped to the individual anatomical situation by bending before initial fitting in order to avoid pressure sores. Four optionally applicable toe correction straps effectively prevent a claw position of the second to fifth toes.

Instruction to use:

Open the support fully in a first step and place it on the ground. Then, place the affected foot on the support so that the side reinforcement splint ends in the front area at the tip of the large toe. With the next step, guide the sealing tape around the metatarsus and close the Velcro fastener using appropriate tension. Here, you must observe that the integrated splay foot pad should be placed as centrally as possible behind the metatarsus heads II-IV.

With the next step, guide the large toe tape around the large toe using appropriate tension and close the attached Velcro fastener.

FR

If desired or prescribed by the doctor, toes II to V can be kept in an upright position using the enclosed toe correction straps. For this purpose, guide the straps around the respective toes and Velcro them on to the lower side of the support using appropriate tension.

Indications:

Hallux valgus, after hallux surgery to ensure the success of treatment, post-operative capsule relief.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

71 % polyethylene, 29 % polyurethane

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. Avoid hindering blood circulation, in case of numbness loosen Velcro closures and remove it if necessary. If complaints continue, please contact the doctor.

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