ES

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EN

Your physician has prescribed you with a high-quality therapeutic shoe, which is suited for early mobilisation in the form of this aid. Please observe the following points in order to ensure that wearing this shoe is as comfortable as possible for you.

Instruction for use:

Important: You are not permitted to drive a vehicle whilst wearing the BORT forefoot relief shoe.

Protect the open foot wound by means of a bandage or similar product. Always wear a non-slip shoe with the same heel height on your healthy foot. Do not wear the shoe directly on the skin.

The supplied toe cap can be adjusted using the Velcro closure located on the shoe in order to protect against the cold or dust if necessary.

Indications

Post-operative, wounds/injuries of the forefoot and the sole of the foot, post-operative following corrective osteotomy in the forefoot area.

Contraindications:

No contraindications are known if used correctly.

Composition:

Sole: 100 % EVA

Upper: 50 % PUR, 35 % polyester, 15 % polyamide

Knit loop / Velcro straps: 100 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

Avoid restricting the circulation of the blood.

Check your foot for pressure points several times a day in order to prevent injuries to the skin.

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