ES

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Sohle: 100% EVA

Obermaterial: 50% PUR, 35% Polyester, 15% Polyamid

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EN

Your physician has prescribed you with a high-quality therapeutic shoe, which is suited for early mobilisation in the form of this aid.

Please observe the following points in order to ensure that wearing this shoe is as comfortable as possible for you.

Instruction for use:

Important: You are not allowed to operate a vehicle when wearing the BORT surgical shoe!

Protect the open foot wound by means of a bandage or similar product. Always wear a non-slip shoe with the same heel height on your healthy foot. Do not wear the shoe directly on the skin.

Indications

Post-operative and other wounds, even when treated with extensive and possibly padded bandaging, where the healing process takes place over an extended period of time and weight can be placed partially on the foot whilst walking, even outdoors. An additional insole is not required.

Contraindications:

No contraindications are known if used correctly.

Composition:

Sole: 100% EVA

Upper: 50% PUR, 35% polyester, 15% polyamide

Knit loop / Velcro straps: 100% polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

Avoid restricting the circulation of the blood.

Check your foot for pressure points several times a day in order to prevent injuries to the skin.

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Composition des matières :

Semelle : 100 % EVA

Recouvrement : 50% PUR, 35% polyester, 15% polyamide

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