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EN

Your physician has prescribed you with a high-quality therapeutic shoe, which is suited for early mobilisation in the form of this aid. Please observe the following points in order to ensure that wearing this shoe is as comfortable as possible for you.

Instruction for use:

Important: Make sure to use the heel zipper to put on and remove the shoe! The surgical shoe's Velcro straps serve only to regulate the width and fit.

Protect the open foot wound by means of a bandage or similar product. If appropriate, make sure to use crutches when wearing the BORT Comfort Surgical Shoe.

1. Open the Velcro straps and the heel zipper before insert your foot into the shoe.
2. Close the heel zipper and then the strap over the mid-foot.
3. Position the center front strap in the middle and fasten it on both sides.

Indications:

Post-operative and other wounds, even when treated with extensive and possibly padded bandaging, where the healing process takes place over an extended period of time and weight can be placed partially on the foot whilst walking, even outdoors. An additional insole is not required.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

Sole: 100 % EVA

Upper: 50 % PUR, 35 % polyester, 15 % polyamide

Knit loop / Velcro straps: 100 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

Avoid restricting the circulation of the blood.

Check your foot for pressure points several times a day in order to prevent injuries to the skin.

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