IT

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EN

The BORT Heel Spur Cushion consists of viscoelastic material. This absorbs the shocks, which could lead to fatigue and cumulative trauma, suppresses them, and could also impede recovery from injuries at the feet, legs and back.

Fitting instructions:

1. Remove plug from the center of the Heel Spur Cushion for pressure relief.
2. Place the Heel Spur Cushion in the heel of the shoe, blue textile-covered side up.
3. Place the second cushion in the shoe of the non-affected foot. Do not remove the plug.

Indications:

Heel spur, Achillodynia, Haglund's deformity, axial shock absorption in the case of leg joint arthrosis.

Contraindications:

Sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

100 % PUR foam

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. BORT Heel Cushions are suitable for all kinds of shoes or boots and are long-lasting with good care. Wipe underside with warm soap water or isopropyl alcohol to restore tackiness.
Do not transfer the BORT Heel Cushion from one pair of shoes to another. Frequent transfers reduce effectiveness.

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