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REF 950 240

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D95024012023-071005 ML 1 Rev.01

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EN BORT Heel Lift

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a shock-absorbing, removable heel lift.

Indications

Compensation of differences in leg-length, Achilles tendon disorders (Achillodynia).

Contraindications

Impaired sensation and circulation in the specific body region, skin diseases in the specific part of the body.

Application risks/Important notes

- This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the medical device, please neither use any creams nor ointments on or around it as they can destroy the material
- please do not subject the medical device to direct heat or temperatures exceeding 90°C (e.g. hot water, vehicles, the sauna, heating) as this may result in damaging the material

Fitting instructions

Place the heel lift in the shoe with the blue, textile-coated side upwards. Ensure that the heel lift is inserted for the leg to be lifted.

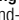
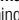
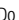
Removal

Remove the heel lift from the shoe at the end of the day and store it at a clean location in order to minimise soiling and unpleasant odours.

Material composition

100% Polyurethane (PU)

Cleaning information

 Hand-washing 30 °C  Do not bleach  Do not dry in a tumble dryer  Do not iron  Do not clean chemically
Do not use fabric conditioner.
If required, you can clean your heel lift with soap and lukewarm water. Dry the heel cushions in the fresh air.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 07.2023

 Medical device |  Single patient – multiple use

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