**MD**
**REF 107 070**

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**EN Heel Cushion with SoftSpot**

**Intended purpose**  
 This medical device is a shock absorber/heel cushion for local pressure relief on the back of the foot during heel impact.

**Indications**  
 Heel spur, achillodynia, Haglund's deformity, shock absorption in the case of arthrosis of the ankle, knee or hip joints.

**Contraindications**  
 Impaired sensation and circulation in the specific body-region, skin diseases in the specific part of the body.

**Application risks/Important notes** △  
 The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used

- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the medical device, please neither use any creams nor ointments on or around it as they can destroy the material

**Fitting instructions**  
 Place the silicone heel cushions in your shoes so that they interlock with the heel area of the shoes. Here, the lateral elevation should point upwards. Please observe the right and left versions here. The left side is marked with an L and the right side with an R.

**Removal**  
 Remove the silicone heel cushion from the shoe at the end of the day and store it at a clean location in order to minimise soiling and unpleasant odours.

**Material composition:** 100% silicone (SI)

**Cleaning information**  
 If required, you can clean your silicone heel cushions with soap and lukewarm water. Dry the heel cushions in the fresh air. If required, the upper side can be lightly powdered.

**Guarantee**  
 The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

**Useful life/Lifetime of the product**  
 The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

**Duty of notification**  
 If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact)

**Disposal**  
 Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

**Declaration of conformity**  
 We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

Status: 05.2023

**MD** Medical device | Single patient - multiple use