# Generation Kniebandage

Knee Support | Band pour le genou | Vendaje para la rodilla | Ginocchiera

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Knee supports retain damaged tissue structures (joints, ligaments, capsules and cartilage). Here, the compression supports the retaining function of the ligaments, thus relieving the joint. At the same time, the BORT Generation knee support helps to avoid incorrect movement to a large extent.

### Instructions for use:

As a first step, the circumferential harness and the velour fastening on the knee support thigh is opened. If required, the harness can be unthreaded because the joint strap should always be located on the outer side of the leg.

Pull the knee support as far as possible over the knee using the two gripping aids so that the silicone pads enclose the kneecaps.

Then close the knee support using the hand straps.

The circumferential harness is then threaded through the strap ring and fixed. Finally, check the fit and correct positioning of the knee support, so that the support supports and compresses but doesn't constrict.

**Indications:**

Chronic post-traumatic or post-operative irritations of knee joint soft tissue, degenerative knee joint diseases, e.g. mild to moderate ligament instability of the knee joint, gonarthrosis, meniscus injuries.

**Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

**Composition:**

58 % polyamide, 40 % elastodines/latex, 2 % spandex

**Important note:**

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and may trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367