Generation Daumen-Handorthese

Thumb-Wrist Brace | Orthèse pour la main et le pouce |
Órtesis para la mano y el pulgar | Ortesi per mano e pollice

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



Wrist braces support damaged tissue structures (joints, ligaments, capsules and cartilages). They support the retaining function of the ligaments, thus relieving the joint. The inserted aluminium splint is shaped to retain the hand in a relaxed, relieving posture.

The BORT Generation Thumb-Wrist Brace was developed for patients who require a brace which is easy to put on, is very comfortable to wear and has a wide area of application.


Instructions for use: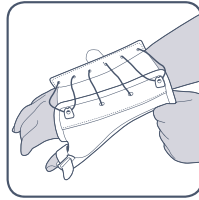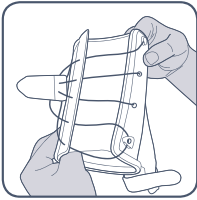

Before putting it on, set the thumb wrap and the wrist fastener to the largest possible circumferences and pull the BORT Generation Thumb-Wrist Brace over the thumb and the metacarpus down to the lower arm.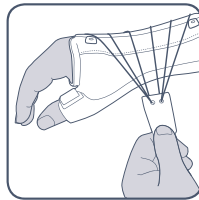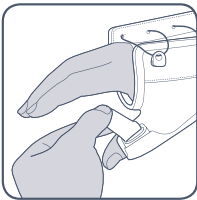

Then, set first the thumb wrap and next the wrist cord strap to the individual width using the Velcro areas.

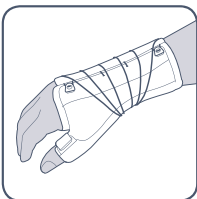Close the brace fully.

If required, the internal aluminium reinforcement splint can be removed from the brace and be individually formed by an expert.

Indications:

Pre-operative, post-traumatic, post-operative conditions, i.e. after thumb ligament reconstruction, immobilisation in the case of rheumatic-inflammatory processes, i.e. rhizarthrosis, in the case of degenerative diseases such as carpal tunnel syndrome.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

84 % polyamide, 16 % spandex

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. Wear splint only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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