Generation Knieorthese

Knee brace | Orthèse du genou | Ortesis de rodilla | Ortesi per il ginocchio

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
Knee joint soft braces support damaged tissue structures (joints, ligaments, capsules and cartilages). Here, the BORT Generation Knee Brace supports the supporting function of the ligaments and stabilises, thus relieving the joint.

Instructions for use:


Before putting the knee brace the side joint splints must be adapted to the contour of the leg by an expert. For forming, remove the splints from the textile and align to the lateral course of the patient's leg by bending it up. Then, on, the desired extension and flexion limitation can be set by a technician if required. For this, the delivered grub screws must be screwed into the corresponding thread hole on the medial and lateral joint splints.

The following restrictions are available:


- Extension: 15°, 75°
- Flexion: 0°, 30°, 55°, 75°, 95°, 110°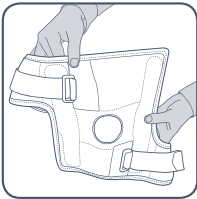


For fitting, leave the two Velcro straps and the BORT Generation Knee Brace padding supports below them open initially.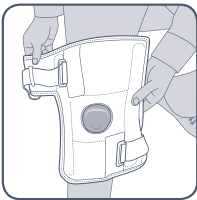

When sitting, lay the knee brace on the stretched knee joint so that the patella recess is directly around the knee-cap.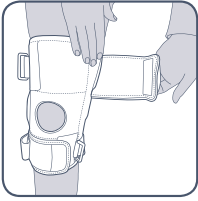

Then guide the padding supports around the thigh and shank and close them.

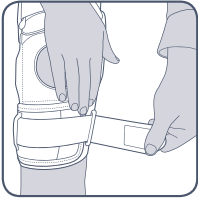Finally guide the two Velcro straps through the guide loops and close them with the desired tension.

Indications:

Degenerative diseases, e.g. mild to moderate lateral ligament instability of the knee joint, gonarthrosis, meniscus injuries.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

84 % polyamide, 16 % spandex

Important Instructions:

No reutilisation – this medical device is meant for the care of one patient only. Wear splint only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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